

**User Guide**

# **G-TRAN**

**The “All-In-One”  
Travel Solution**



**Guidosimplex**

[www.guidosimplex.it](http://www.guidosimplex.it)

**Dear Customer,**

*Thank you for having purchased a Guidosimplex device,*

*As with all of our products, this device, was designed and created respecting the highest possible levels of quality and safety standards.*

*All Guidosimplex products are manufactured from the best materials available on today's market , this means that when you buy a Guidosimplex product you buy quality and reliability, which results in less maintenance.*

*Guidosimplex products coexist in perfect harmony in parallel with the original commands and design of the vehicle and conventional driving can be restored on command.*

*This device is capable of fulfilling all needs as per design as long as a correct usage is performed.*

**Therefore we advise you to consult the below reported user guide prior to using the device.**



## OPERATING THE SWIVEL BASE





To rotate the seat inwards press (A) To rotate outwards press (B)

# RAISING AND LOWERING THE SEAT



## TRANSFERRING THE SEAT

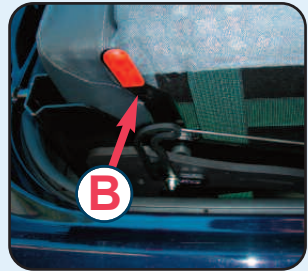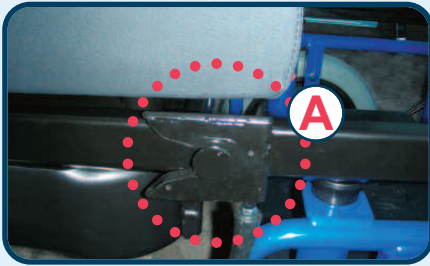

release seat



The operation required to transfer the G-TRAN – E seat from the vehicle to the powered chassis is simple and safe for both the End User and the Assistant. Using the hand control unit, position the seat correctly in line with the powered chassis. This will allow the chassis to dock smoothly (A) and safely to the seat base. Use the dedicated seat release lever (B) to free the seat and allow it to slide smoothly until it is correctly in position on the powered chassis.



**In the case of malfunctioning or any other problem related to the device in question please contact the nearest authorised Guidosimplex dealer.**

**[www.guidosimplex.it](http://www.guidosimplex.it)**

This user guide was printed by Guidosimplex S.r.l. and cannot be considered a valid warranty document.

Guidosimplex reserves the right to update without notice the document in question.

Reserved Copyrights. Guidosimplex S.r.l.



**Guidosimplex S.r.l.**  
**Via del Podere San Giusto, 29**  
**00166 Roma**  
**Tel. 06 6152801 r.a.**  
**Fax 06 61528036**  
**infoclient@guidosimplex.it**  
**www.guidosimplex.it**

